0476

RI.MOS. s.r.l. viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 25755
fax +39 0535 25509 - e-mail: info@rimos.com – www.rimos.com

FERTIRAM®

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Disposable sterile devices for assisted fertilization

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Needle for ovum pick-up and follicle flushing

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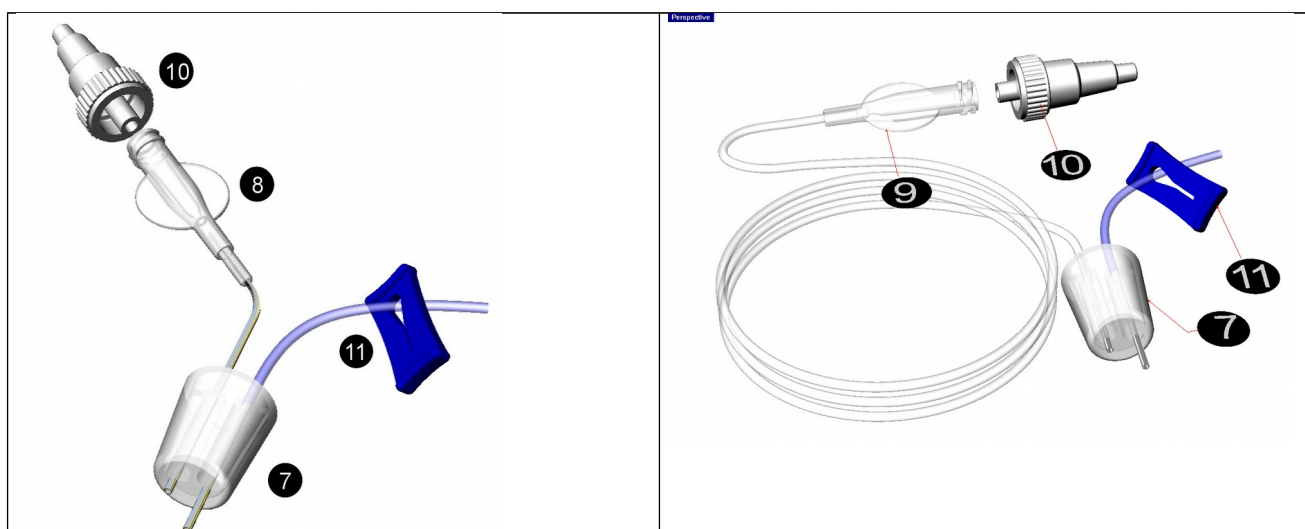
GB

Instructions for use

Prepare the vaginal area of the patient and the sterile field. Use a vaginal probe-cover to protect the ultrasound probe and connect it to a needle guide. Make sure that connection is well secured. Introduce the ultrasound probe in the vagina to confirm follicle persistence and accessibility. Under ultrasound control, introduce DOUBLECYTE or DOUBLECYTE-N in the guide-needle connected to the ultrasound probe. Push the needle through the vaginal wall firmly until the ovarian stroma and the follicle are reached. Gently aspirate one follicle after the other by controlled vacuum pump. Perform follicle flushing through the second way of the system that is provided with a stopcock. At the end of the procedure remove the ultrasound probe and the needle. Use of DOUBLECYTE rather than DOUBLECYTE-N and vice versa, depends on the more suitable needle connection to the aspiration pump. Within DOUBLECYTE, a female luer-lock connector (picture a), is directly connected to the pump tubing. Within DOUBLECYTE-N, the needle is connected to the pump by a 80/100 cm extension tubing and a female luer-lock connector (picture b). Both versions are supplied with an additional adapter (10) allowing multiple connections.

Caution: Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

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FERTIRAM products are manufactured from biocompatible materials suitable for human assisted fertilization, MEA & LAL tested and Beta Sterilized.