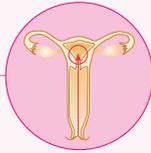
Embryo transfer

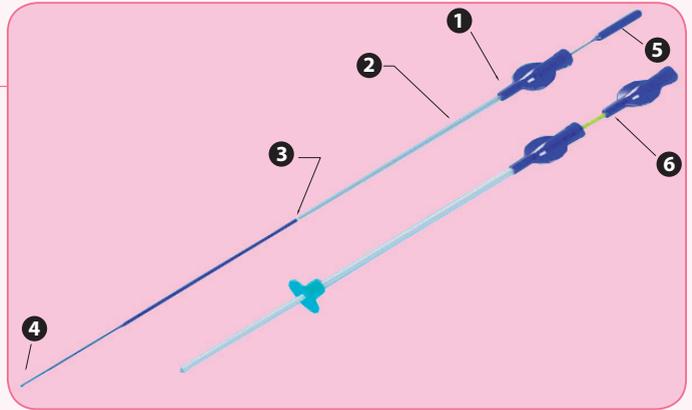
TRANSFER SLOW

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Embryo transfer set with introducer (without syringe)

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For loading and transferring the embryos into the uterine cavity in a very gentle and atraumatic way.

The catheter is composed by:

- Female connector serving as finger-grip (1);
- Rigid, centimeter-marked proximal portion (2);
- Semi-rigid & transparent centimeter-marked medial portion, to enhance visualization during embryo positioning (3);
- Soft & transparent distal portion for atraumatic penetration into the uterine cavity (4);
- Centimeter-markings and radiopaque line placed all along the catheter
- Centimeter-marked stylet acting as a microplunger grants a very slow & gentle embryo aspiration through the medial portion (5);
- Stylet/microplunger blockage to avoid the risk of embryo loss .

This set includes also a **GUIDING CATHETER (6)** that facilitates the insertion of transfer catheter. It is provided with a sliding cursor and with a centimeter markings on the tip.

This system avoids the use of the syringe during embryo aspiration/expulsion phases. This largely excludes the risk of excessive pressure and turbulence to the embryos particularly during expulsion phase and makes the procedure definitely less traumatic.

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12 pcs. per box.

Single packaged, single use, Beta sterilized and CE marked.

Cod.	Descrizione - Description
770250	Ø porzione distale morbida 1,33 mm (4 Fr) soft distal portion O.D. 1.33 mm (4 Fr) lung. totale 260 mm - total length 260 mm